

ACCESSORIES: INSTALLATION: TECHNICAL DATA: TERMINATION:-CLAMPING SLEEVES:-WARNING: NOMINAL OPERATING VOLTAGE 160KV DC STANDARD FLANGE T20131/A CABLE SHOULD ONLY BE FITTED BY A TRAINED PERSON.
OIL MUST NOT BE USED ON THE RUBBER TERMINATION, ONLY SUITABLE HIGH VOLTAGE GREASE. - USING AN ESSEX X-RAY APPROVED RECEPTACLE CABLE LENGTH TOLERANCE +/- 100mm QUICK LOCK T20096/B ENSURE BY ADJUSTING THE CLAMPING SLEEVE THAT WHEN THE TERMINATION IS FULLY INSERTED IN THE SOCKET THERE IS A ANDREX TYPE FLANGE T20045/C CABLE:- VOLTAGE WITHSTAND GAP OF 6mm BETWEEN THE MATING FACES OF THE CLAMPING SLEEVE 50KV (RMS) AC A INSTALLATION GREASE P00075 AND SOCKET BEFORE TIGHTENING THE CLAMPING SLEEVE. FOR FULL CABLE DATA SEE ESSEX CABLE DATA SHEET A

ESSEX

Dandong Light Electric Co.,Ltd Rm.1606 Jiadi Square,No.64,Binjiang Middle Rd, Zhenxing, Dandong, Liaoning China T:0415-6123232 F:0415-6123533

	REVISIONS		DUE TO C	ONTINUING F	PRODUCT DI	EVELOPMENT	ESSEX	X-RAY	RESERVES THE RIGHT TO MODIFY DATA	WITHOUT NOTICE	
NO.	DATE	BY	DRAWN	APPROVED	DATE	ISSUE	SCALE		O NOT TO BE REPRODUCED IN WHOLE OR PAR	T WITHOUT	
4	02.01.01	DCN 1405	JDF		15.09.95	7		N.T.S.	PRIOR PERMISSION OF ESSEX X-RAY & MED	RAY & MEDICAL EQUIPMENT LTD.	
5	11.05.01	DCN 1455	TITLE	TITLE 160KV FURO RICHT ANGLE (ROA TYRE)							
6	13.07.01	DCN 1534	160KV EURO RIGHT ANGLE (R24 TYPE) (CABLE TYPE C2042)							C1001	
7	06.03.09	DCN 2905								31004	