

TECHNICAL DATA:	ACCESSORIES:	INSTALLATION:		
TERMINATION:- NOMINAL OPERATING VOLTAGE 100KV DC	CLAMPING SLEEVES:- 100KV FLANGE (SMALL) T20051/A	WARNING:		
-USING AN ESSEX X-RAY APPROVED RECEPTACLE CABLE LENGTH TOLERANCE +/- 100mm	100KV FLANGE (LARGE) T20052	CABLE SHOULD ONLY BE FITTED BY A TRAINED PERSON. OIL MUST NOT BE USED ON THE RUBBER TERMINATION, ONLY SUITABLE HIGH VOLTAGE GREASE.		
CABLE:- VOLTAGE WITHSTAND 100KV DC 30KV (RMS) AC	INSTALLATION GREASE P00075	ENSURE BY ADJUSTING THE CLAMPING SLEEVE THAT WHEN THE TERMINATION IS FULLY INSERTED IN THE SOCKET THERE IS A GAP OF 4mm BETWEEN THE MATING FACES OF THE CLAMPING SLEEVE AND SOCKET BEFORE TIGHTENING THE CLAMPING SLEEVE.		
FOR FULL CARLE DATA SEE ESSEX CARLE DATA SHEET A				

ESSEX

Dandong Light Electric Co.,Ltd Rm.1606 Jiadi Square, No.64, Binjiang Middle Rd, Zhenxing, Dandong, Liaoning China T:0415-6123232 F:0415-6123533

	REVISIONS			8	DUE TO C	ONTINUING	PRODUCT D	EVELOPMEN	T ESSEX	X-RAY	RESERVES THE RIGHT TO MODIFY	DATA WITHOUT NOTICE
	NO.	DATE		BY	DRAWN	APPROVED	DATE	ISSUE	SCALE		O NOT TO BE REPRODUCED IN WHOLE	OR PART WITHOUT
	4	02.01.0	01	DCN 1405	JDF		04.01.96	7		N.T.S.	PRIOR PERMISSION OF ESSEX X-RAY	' & MEDICAL EQUIPMENT LTD.
	5	11.05.0			TITLE		DL		10010	(P10)		DRAWING No.
	6	30.03.0		DCN 2282			PHILIPS 100KV (CABLE TYPE (12)	DC1007
	7	06.03.0	09 2	DCN 2905	(CADLE TIPE C					CZZ	12)	1007