

APPROXIMATELY ONE WEEK. ADJUST AS ABOVE IF REQUIRED.

TECHNICAL DATA:			ACCESSORIES:				INSTALLATION:				
TERMINATION:— ONLY ESSEX X-RAY APPROVED RECEPTACLES TO BE USED. <div>⚠</div> FOR FULL CABLE DATA SEE ESSEX CABLE DATA SHEET			INSTALLATION GREASE P00075				WARNING: CABLE SHOULD ONLY BE FITTED BY A TRAINED PERSON. CABLE CAN RETAIN ELECTRICAL CHARGE. DISCHARGE TO EARTH BEFORE HANDLING. A SUITABLE HIGH VOLTAGE GREASE SHOULD BE APPLIED TO THE TERMINATION. ENSURE THAT THE TERMINATION IS FULLY INSERTED IN THE SOCKET BEFORE ADJUSTING CLAMPING FLANGE.				
<div>ESSEX</div> Dandong Light Electric Co.,Ltd Rm.1606 Jiadi Square,No.64,Binjiang Middle Rd, Zhenxing, Dandong, Liaoning China T:0415-6123232 F:0415-6123533			REVISIONS		DUE TO CONTINUING PRODUCT DEVELOPMENT ESSEX X-RAY RESERVES THE RIGHT TO MODIFY DATA WITHOUT NOTICE						
			NO.	DATE	BY	DRAWN	APPROVED	DATE	ISSUE	SCALE	© NOT TO BE REPRODUCED IN WHOLE OR PART WITHOUT PRIOR PERMISSION OF ESSEX X-RAY & MEDICAL EQUIPMENT LTD.
			2	21.12.00	DCN 1409	MGM		29.02.00	4	N.T.S.	
			3	07.12.06	DCN 2590	TITLE FITTING INSTRUCTIONS 100KV TERMINATION					
			4	09.03.09	DCN 2905						
					DRAWING No. DS1044						