



| TECHNICAL DATA: | ACCESSORIES: | INSTALLATION: |
|--|---|--|
| <p>TERMINATION:-</p> <p>NOMINAL OPERATING VOLTAGE 210KV DC</p> <p>- USING AN ESSEX X-RAY APPROVED RECEPTACLE</p> <p>CABLE LENGTH TOLERANCE +/- 100mm</p> <p>CABLE:- VOLTAGE WITHSTAND 250KV DC 75KV (RMS) AC</p> <p>FOR FULL CABLE DATA SEE ESSEX CABLE DATA SHEET</p> | <p>CLAMPING SLEEVES:-</p> <p>STANDARD FLANGE T22341</p> <p>INSTALLATION GREASE P00075</p> | <p>WARNING:</p> <p>CABLE SHOULD ONLY BE FITTED BY A TRAINED PERSON.</p> <p>CABLE CAN RETAIN ELECTRICAL CHARGE.</p> <p>DISCHARGE TO EARTH BEFORE HANDLING.</p> <p>OIL MUST NOT BE USED ON THE RUBBER TERMINATION, ONLY SUITABLE HIGH VOLTAGE GREASE.</p> <p>ENSURE BY ADJUSTING THE CLAMPING SLEEVE THAT WHEN THE TERMINATION IS FULLY INSERTED IN THE SOCKET THERE IS A GAP OF 6mm BETWEEN THE MATING FACES OF THE CLAMPING SLEEVE AND SOCKET BEFORE TIGHTENING THE CLAMPING SLEEVE.</p> |

ESSEX

Dandong Light Electric Co.,Ltd
Rm.1606 Jiadi Square,No.64,Binjiang Middle Rd,
Zhenxing, Dandong, Liaoning China
T:0415-6123232
F:0415-6123533

REVISIONS

| NO. | DATE | BY |
|-----|------|----|
| | | |
| | | |
| | | |
| | | |

DUE TO CONTINUING PRODUCT DEVELOPMENT ESSEX X-RAY RESERVES THE RIGHT TO MODIFY DATA WITHOUT NOTICE

| DRAWN | APPROVED | DATE | ISSUE | SCALE |
|-------|----------|----------|-------|--------|
| MJT | | 06.01.12 | 1 | N.T.S. |

© NOT TO BE REPRODUCED IN WHOLE OR PART WITHOUT
PRIOR PERMISSION OF ESSEX X-RAY & MEDICAL EQUIPMENT LTD.

TITLE

210 SEIFERT SPRING LOADED (R30 TYPE)
STRAIGHT (CABLE TYPE C2236)

DRAWING No.

DS1086